

Invisible to  
a beautiful smile

## Bonding process of attachments



1. Check the position of the attachment aligner. If attachments are to be bonded in several quadrants, cut the attachment aligner into quadrants so that you can carry out the following bonding process in each quadrant.



2. Clean the first quadrant and fill the protrusions of the attachment aligner with a light-curing resin. Press the bonding material accurately into the protrusions.



3. Dry the teeth and apply primer onto the teeth. Position the aligner and slightly press it on everywhere.



4. Completely light cure the resin in accordance with the manufacturer's instructions for use and remove the aligner. Then repeat this process (points 2 to 4) with the other quadrants.



**World Class Orthodontics**®

Ortho Organizers® GmbH

Lauenbühlstraße 59 · D-88161 Lindenberg  
Telefon: +49 8381 890950 · Fax: +49 8381 8909530  
E-Mail: [info@w-c-o.de](mailto:info@w-c-o.de) · Internet: [www.w-c-o.de](http://www.w-c-o.de)